

Assembly Instructions

3 Section King Size Screen

Cat. No. 153950, 153961

1. Assemble the horizontal tubes between the base frames.

Figure #1

2. Slide the lower u-shape frame up through the outside half of the hinge on the vertical tube. Repeat procedure for the opposite side. Figure #2

3. Slide the vertical tubes through the clearance holes at the top of each base frame until the bottom rests on the lower tube of the base frame. Hinges must face opposite direction of each other. Secure with machine screws. Insert casters on model 153961. Figure #3

4. Position the top half u-shape frame so that both the upper and lower hinge face the same direction. Guide the assembled top half u-shape frame onto the bottom section. Push the appropriate top tube into the bottom tube until all tubes are firmly secured. Figure #4

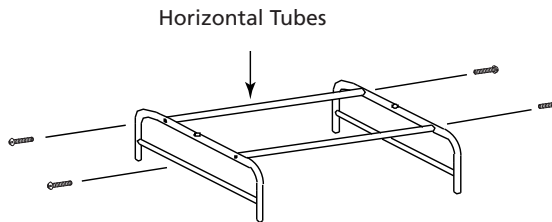


Figure #1

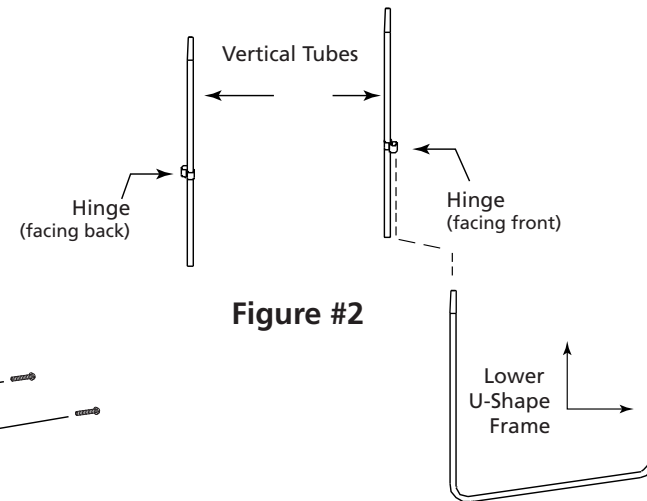


Figure #2

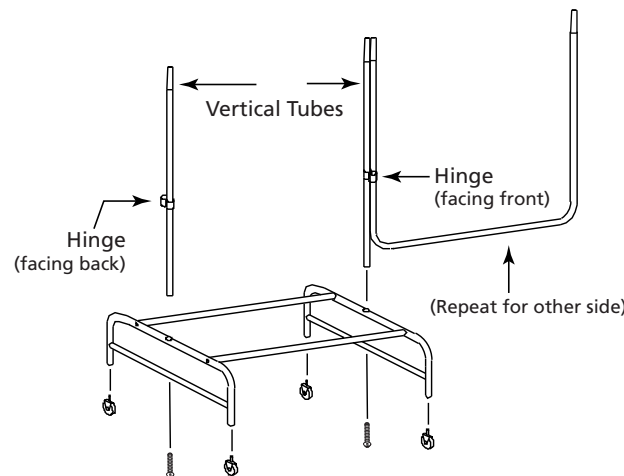


Figure #3

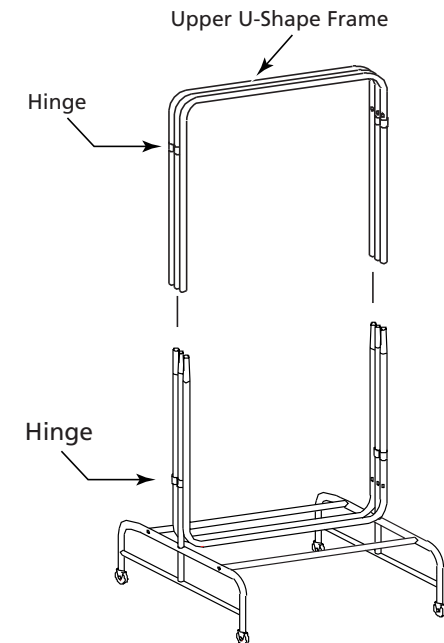


Figure #4



800 Glen Avenue, Moorestown, New Jersey 08057

1.800.257.BEAM (2326) • Local: 1.856.359.2231

Fax: 1.856.359.2249 • E-mail: info@omnimedbeam.com

Visit our web page on the internet: www.omnimedbeam.com

Products to Help People...Help People

5. Unfold vinyl panels (insert rods-top & bottom of panels).
 King Screen Models have 2 vinyl panels per section.
 After all screens have been assembled (**Figure #5**), snap
 the top and bottom screen panel rod into each section.

King Screen Models Described Above Are Now Ready For Use.

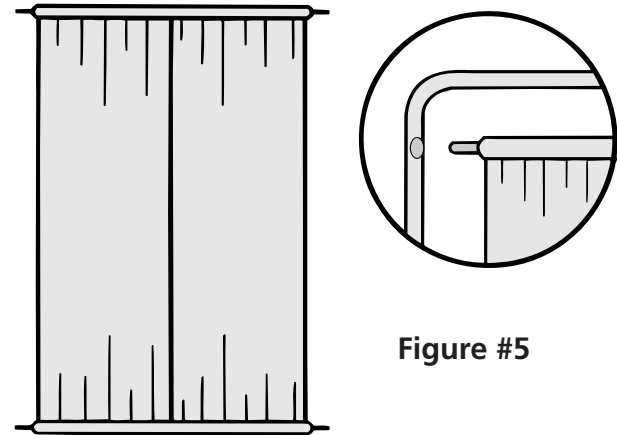








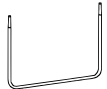


Figure #5

Parts List

3 Section			3 Section		
	Vinyl Panel	6		3 PC. Upper U-Shape Frame Assembly	1
	Screen Rod	6		Base Frame	2
	*Caster	*4		Base Frame Horizontal Tube	2
	Machine Screw	6		Vertical Tube	2
		6		Lower U-Shape Frame	2

*If ordered as Screen with Casters