


Instruction Sheet

Installing Spacer Bars/Stacking

Cat No. 264004-10

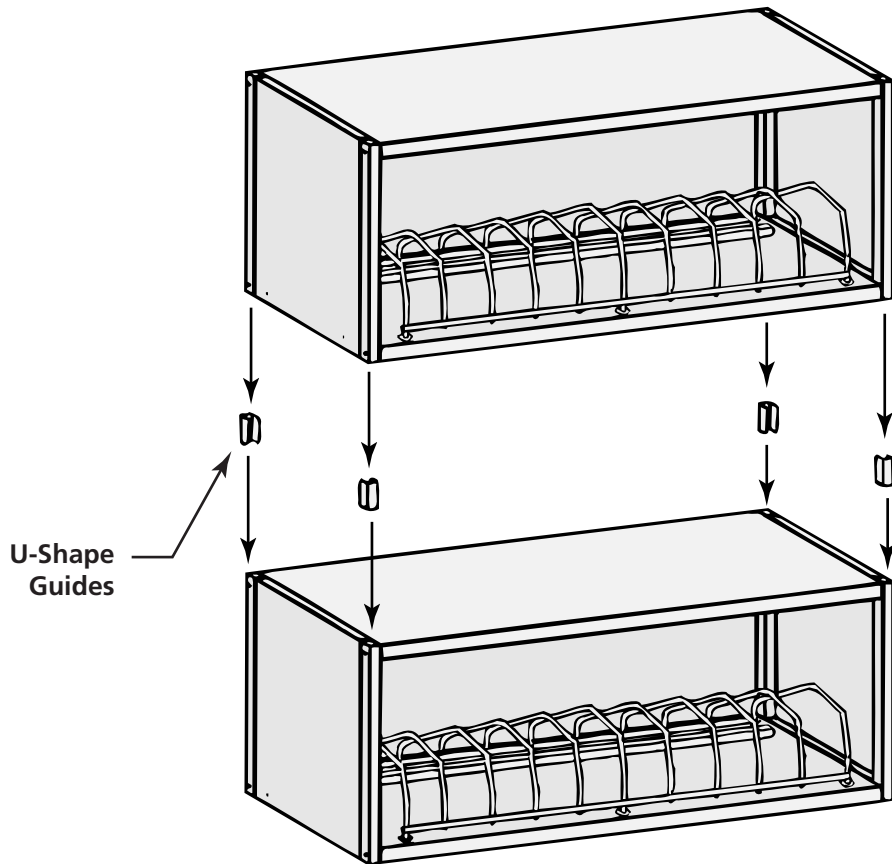

U-Shape Guides



U-Shape Guides
(4 Provided)

Spacer Bar

Used to position side open style binders toward front of shelf.
Remove Spacer Bar when using longer, top open style binders

To Install Spacer Bars

1. Before stacking cabinets together, place units on their sides.
2. Remove front wire organizer nuts. Put aside for reassembly. Lift up wire organizer and slip spacer bar through.

Place the threaded studs (attached to the Spacer Bar) into the two holes provided in the shelf toward the back of the cabinet. Secure the Spacer Bar to the shelf using the two 1/4" lock washers and nuts.

3. Reattach wire organizer to shelf with 1/4" lock washer and nuts. (Reverse instructions to remove Spacer Bars when Spacer Bars are not required.)

Stackable Instructions For Table Top Storage Rack

1. Set aside cabinet that will be the top unit.
2. Remove plastic plugs from the top of each corner post on lower unit.
3. Insert u-shape guides in the top of each corner post as shown in the diagram. Place the radiused sides to either the front or back of cabinet.
4. Remove the plastic plugs from the bottom of each corner post on the cabinet that will be the top unit.
5. Lower the top cabinet unit into place onto the u-shape guides.
6. Store the corner post plastic plugs for possible future use as single units.

*Clean with mild soap and water



✉ 800 Glen Avenue, Moorestown, New Jersey 08057
☎ 1.800.257.BEAM (2326) • Local: 1.856.359.2231
📠 Fax: 1.856.359.2249 • E-mail: info@omnimedbeam.com

Visit our web page on the internet: www.omnimedbeam.com

Products to Help People...Help People