

Instruction Sheet

EVO Articulating Informatics Work Station #291556-EVO

1) To begin installation of your wall desk open the door using the default combination: push buttons 2 and 4 simultaneously followed by button 3, then turn the knob clockwise. Incorrect keystrokes can be cleared by turning the knob counter clockwise. Directions for setting a new combination are on the reverse.


2) Use the four keyhole slots in the rear and appropriate hardware to fasten the cabinet securely to wall studs.

3) Attach your monitor to the monitor plate using the screws provided. For 100 mm mounting pattern monitors, use the outer set of holes with the black spacers, for 75 mm patterns use the inner holes and omit the spacers. Slide the monitor and plate onto the tilter head. Loosen the four screws holding the keyboard plate assembly and position it at an appropriate height for your monitor and tighten the screws. Remove the adhesive backing from the four Dual Lock reclosable fastening strips and use them to fasten the keyboard to the keyboard plate.

4) Use the included hex key to adjust the monitor tilt tension and the counterweight force.

5) Position your CPU on the riser brackets with the connectors facing downward. Adjust the riser brackets if necessary to prevent any interference with CPU cables. Remove the adhesive backing from both sides of one of the dual lock strips and use the strip to secure the CPU to the rear wall.

Your desk is fully assembled and ready for use.

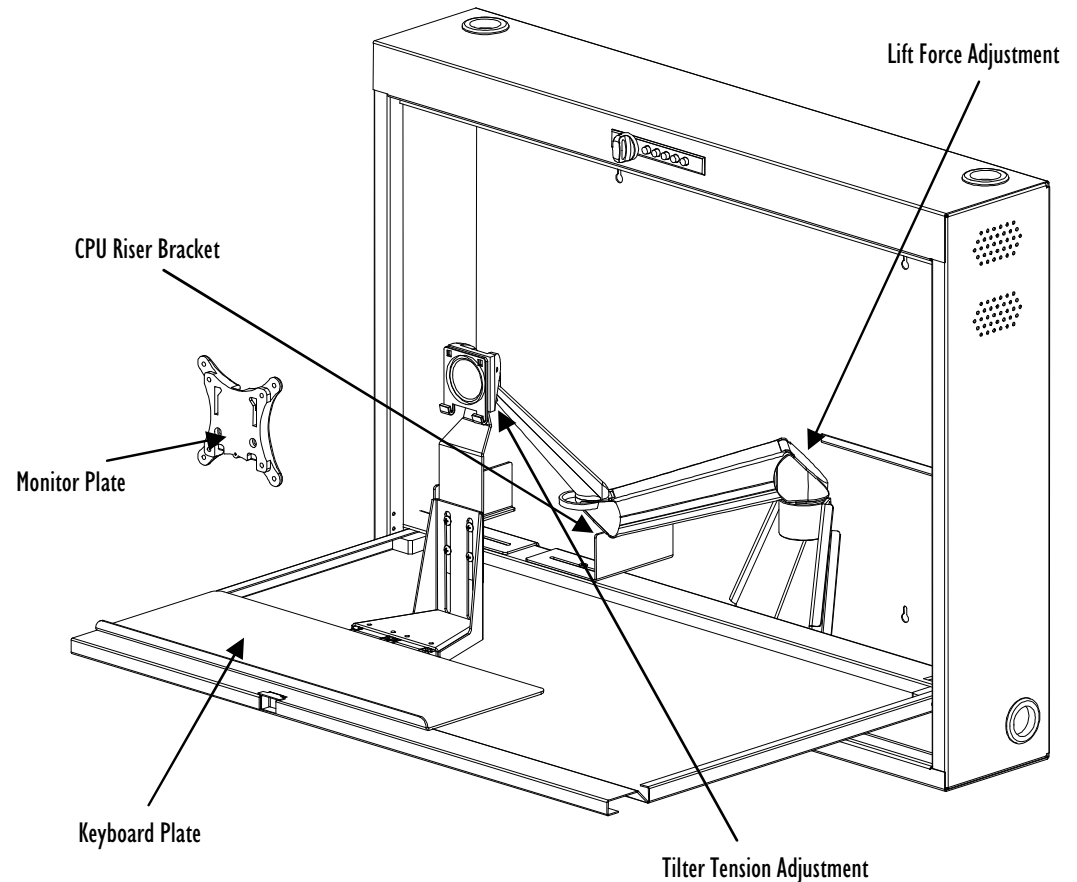


MANUFACTURERS OF **IBeam**® PRODUCTS
OMNIMED INC.®
800 Glen Avenue • Moorestown, N.J. 08057
800.257.2326 • FAX: 856.359.2249
www.omnimedbeam.com

Instruction Sheet

Combination Lock Reset

Open the cabinet using the current combination. Again, press the combination code into the lock. Find the combination reset button and lever on the rear of the lock and press it in. Clear the existing combination from the lock by turning the knob counter clockwise. Key in your new combination. Two or more buttons can be pushed at once, but no button can be used more than one time. Turn the knob clockwise to set the combination.

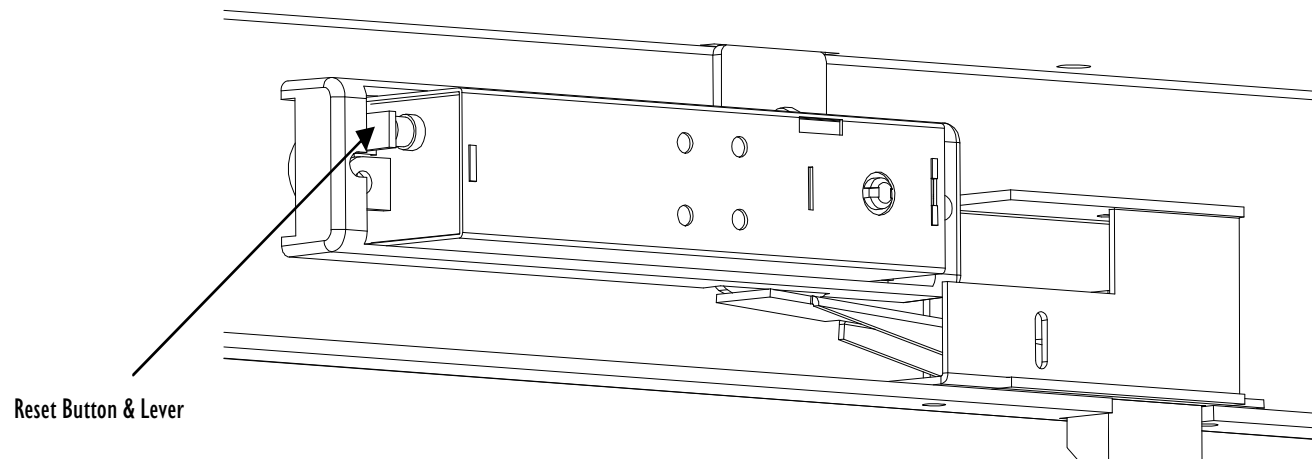

MANUFACTURERS OF **IBEAM**® PRODUCTS
OMNIMED INC.®
800 Glen Avenue • Moorestown, N.J. 08057
800.257.2326 • FAX: 856.359.2249
www.omnimedbeam.com